

3Shape Trios

1. Search 'Splint King' or info@splintking.com in your Trios machine.
2. Once found, click and add connection.
3. Once we are notified, a connection will be established, and jobs will be able to be sent directly from your scanner.

Medit Link

1. Open your Medit Link account.
2. Go to the 'partners' tab and search 'Splint King'
3. Once found press 'request partnership'.
4. Your practice will be notified once a connection is established and jobs can be sent directly from you Medit scanner.

Sirona Connect (Omniscam/Primescan users)

1. Log into your Sirona connect portal.
2. Search for 'Splint King' and request to add connection.
3. Email notification will be sent once connection has been established.

Analogue Impressions

1. Upper PVS impression with heavy body and light body.
2. Lower PVS impression with heavy body.
3. Bite registration.

For any other scanner systems or for more information, please contact us at info@splintking.com with your practice details so we can arrange the most efficient method of job transfer.